



Magic Guide System



C.P.C TECHNIQUE (Crestal approach with Periosteum Control)

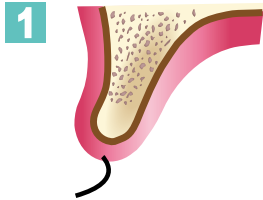


1 Make a mark on the soft tissue for of placement site.

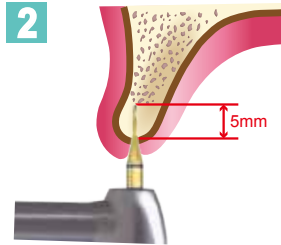


2 Perform drilling up to a depth of 5mm in the direction of planned fixture placement.

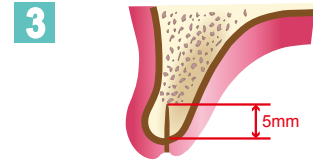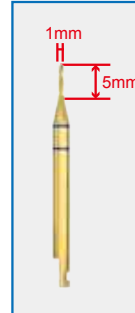



3 A 5mm hole is formed by the Pin Drill.

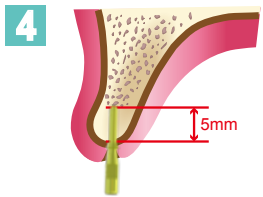


4 Select guide pin that matches the magic drill suitable for the diameter of the fixture to be placed. Place the guide pin in the hole formed by the Pin Drill and take CT scans.

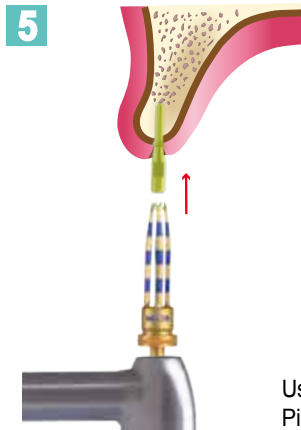


5 Use Magic Drill that matches the Guide Pin that was placed.

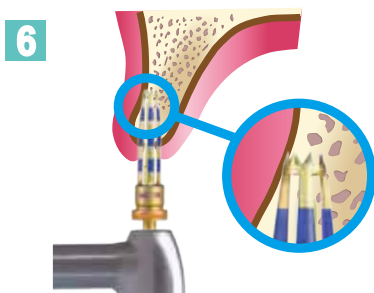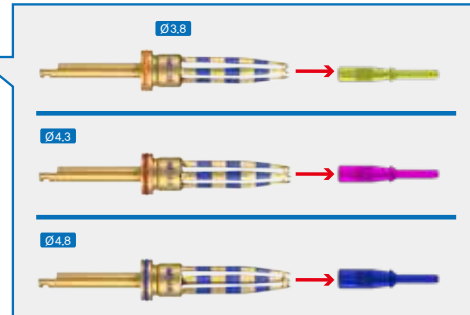



6 Perform drilling up to the depth of the guide pin.

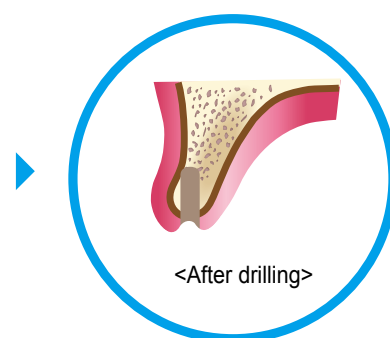



Magic Guide System

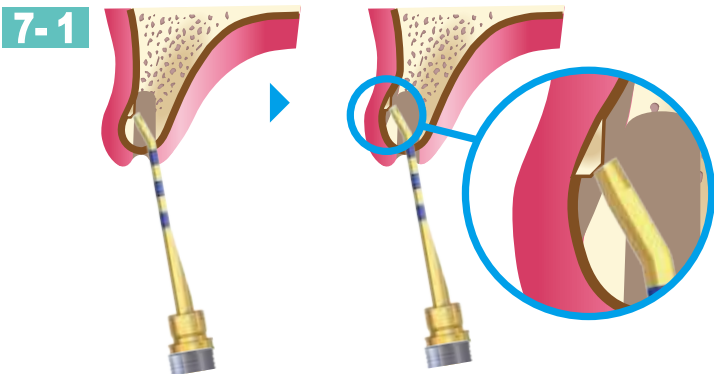

Fracture the surrounding bone of the hole using the Periosteum Lifter and detach the periosteum from the alveolar ridge.

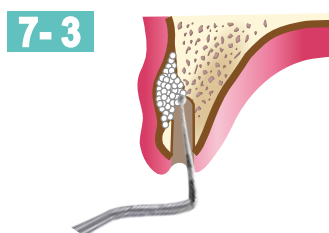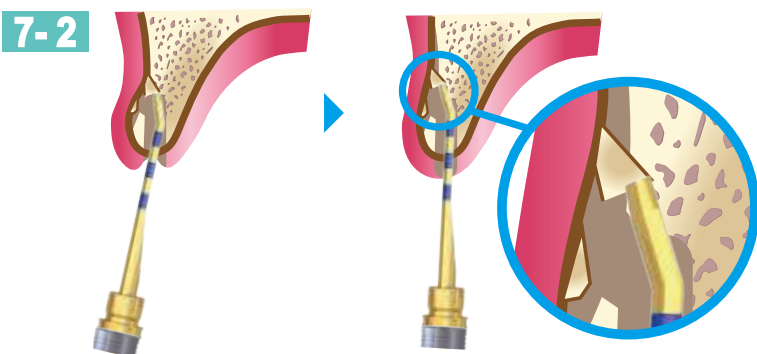

Perform GBR procedure

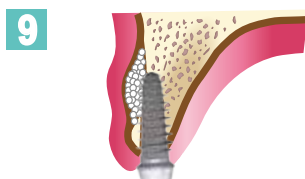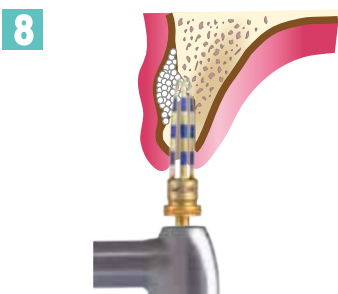

Perform drilling & place MAGICORE fixture.